

# Practical Guide for Usage of *Skintestor OMNI™*

## For practical usage, follow the steps below\*:

- Match arrow on the Skintestor OMNI™ device to the arrow on the tray
- Insert device into the tray and remove
- Hold the device perpendicular to the skin with arrow pointing toward the patient, allowing for easy location of each site
- Press down with moderate pressure
- Without lifting, rock the device
  - On arm: rock the device from side to side
  - On back: rock the device from side to side, then top to bottom
- Withdraw the Skintestor OMNI (When properly performed, the test will leave visible marks on the skin without causing bleeding)
- Discard the used device into a suitable biohazard container
- Repeat steps until testing is complete



Based on an in-house study,  
GREER® recommends filling wells  
with 0.250 mL of antigen†

\*Skintestor OMNI Directions for Use.

†Data on file. GREER.

Skintestor OMNI™ is manufactured by QTI Corporation and distributed by GREER®.